

Seeing more in one view means less refocusing

Refocusing interrupts your procedure. So we've equipped the PROvido with our innovative FusionOptics technology, 300 W xenon light and Small Angle Illumination (SAI). FusionOptics unites high resolution with an increased depth of field for a greatly expanded area in full focus. When coupled with a concentrated xenon light beam, you benefit from a deeper view into cavities with more detail and less shadow.









Rear fine focus




SpeedSpot laser focus

The view you need, fast

Meet the visualization needs of your procedure and your team at every moment with optics that are fast to set up and adjust.

- > Start fast with the SpeedSpot system that uses two laser beams as a focusing reference to quickly provide a defined focus point for all three viewing positions (surgeon, assistant, camera)
- > Quickly achieve maximum magnification with the optional Magnification Multiplier that boosts magnification by 40% in an instant
- > Give your opposite assistant viewing flexibility with an independent fine focus

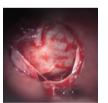

Without SAI



With SAI

Deeper insights

Small Angle Illumination (SAI) technology and bright 300-Watt xenon light provides a concentrated light beam. You benefit from a bright view with less shadow right down to the bottom of deep, narrow cavities.

GET INTO POSITION & GET GOING

Quickly and effortlessly get into your ideal operating position and start working.



Compact and fully integrated

Take control

Intuitive touch-screen control panel for fast set up of all microscope functions and storage of customized profiles.

- > Flexible intra-operative control of key functions via handgrip or optional wireless foot switch
- > Full integration of HD video camera means that recording and image capture is also just a tap away





Full range of movement and tilt of the optics carrier

Positioning was never so easy

Positioning the optics carrier is fast and almost effortless. This not only limits strain from harsh movements, but also makes set-up highly efficient.

- > Electromagnetic brakes and AC/BC balancing for effortless, balanced positioning, whatever the required angle
- > Robust, full-metal stand ensures fast stabilization and remains exactly where needed
- Achieve highly precise micro adjustment with just a touch of the XY joystick control













A clear view in the OR

Share your view with your team to enhance workflow efficiency and teaching.

- > Stand-mounted monitor arm for flexible positioning
- > 24" HD monitor and 27" HD touch-screen monitor available

Record at the touch of a button

The HD C100 imaging and recording system is fully integrated inside the microscope.

Seamless integration allows you to start recording at the touch of a button to capture high definitition surgical footage or still images

Share and teach

Easily save surgical images and footage to patient files and share with students to support your teaching program.

> Simply save to USB or directly to the hospital network via cable for storage in patient files or ready to share with your students


A LEADING LIGHT IN PATIENT SAFETY

Smart illumination minimizes interruption and supports patient safety



Optimal field of illumination

- Autolris automatically adjusts the diaphragm in line with the zoom, so as the field of view decreases so does the field of illumination, ensuring that only the visible area is illuminated
- This prevents the possibility of drying or burning exposed tissue outside the field of view

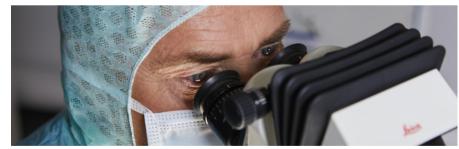

Optimal light intensity

- > BrightCare Plus automatically adapts light intensity to the working distance for safer illumination (up to 60 % light reduction)
- An internal luxmeter provides real-time data so light intensity can be measured from actual bulb output
- Extremely efficient light transmission means that even with less intensity you still have all the light you need

Reliability built in

The PROvido features a bright 300-Watt xenon arc-lamp and back-up 75-Watt LED or 300-Watt xenon arc-lamp with independent lamps and boards.

SPECIFICATIONS

OPTICS AND ILLUMINATION

| OPTICS AND ILLUM | OPTICS AND ILLUMINATION | |
|-----------------------------------|--|--|
| FusionOptics | For increased depth of field and high resolution for main surgeon | |
| Fully apochromatic optics | For high contrast and natural colors without chromatic aberrations | |
| Magnification | 6:1 zoom, motorized Optional 1.4× magnification multiplier | |
| Focus | Motorized via multifocal lens, with manual adjustment | |
| Objective / working distance | 225–600 mm, motorized multifocal lens, continuously adjustable and manual adjustment option | |
| Eyepieces | Wide-field eyepieces for persons wearing glasses 8.3×, 10× dioptric adjustment, ±5 diopter settings and adjustable eyecup | |
| Integrated 360° rotatable adapter | For main surgeon binocular (IVA, ULT) and opposite assistant (ULT) | |
| Illumination | Single Xenon 300-Watt arc-lamp as main illumination with LED lamp as back up or optional dual Xenon arclamp illumination system Continuously variable illumination field diameter Continuously adjustable brightness at constant color temperature | |
| SpeedSpot | Laser focusing aid for fast and exact positioning of the microscope | |
| Autolris | Built-in automatic zoom-synchronized illumination field diameter, with manual override and reset feature | |
| BrightCare Plus | Safety function through working distance- dependent limitation of the brightness, controlled by built-in luxmeter | |
| | | |

CONTROL

| Control unit | Programmable touch-screen with user-friendly Graphical User Interface for control of the microscope and stand Built-in electronic auto-diagnosis and user support Software independent hard keys for illumination Indicator for main/backup illumination and fluorescence modes |
|------------------|--|
| Control elements | Pistol handle with 10 programmable functionsOptional 12-function wireless footswitch |
| IR sensor | For remote control of the external Leica HD C100 camera |

OPTIONS

| UPTIONS | |
|--------------------------|---|
| ULT530 | Full stereo view for main surgeon and opposite assistant, semi stereo view for 2 side assistants Optional integrated HD Camera (HD C100) |
| FL800 ULT | ULT with the Leica FL800 vascular fluorescence observation filter module |
| FL560 | Leica FL560 fluorescence observation filter module |
| IVA530 | Full stereo view for main surgeon, semi stereo view for 2 side assistants and C-mount interface for camera (HD or SD) |
| Integrated documentation | Prepared for integration of video camera system and digital recording. Open architecture |
| Digital Recording | Record still pictures and videos in standard definition (SD) and high definition (HD) using a digital recording device (documentation system) |
| Laser compatibility | compatible with Lumenis AcuSpot 712L Laser Manipulator and Digital AcuBlade |
| | |


CONSTRUCTION


| Base | 700 × 700 mm with four 360° rotating castors with a diameter of 126 mm each and integrated brakes |
|-----------|---|
| Materials | All solid metal construction |
| Load | Max. 8.5 kg from microscope dovetail ring interface |
| Weight | Approx. 350 kg without load |

MANEUVERABILITY

| XY speed | Zoom linked XY speed |
|---------------------|---|
| XY range | 62 x 62 mm |
| Balancing | Manual balancing at microscope carrier and at swing arm |
| Brakes | Floor stand with 6 electromagnetic brakes |
| Carrier for monitor | 610 mm long and flexible arm with 180° rotation and inclination to carry optional video monitor |



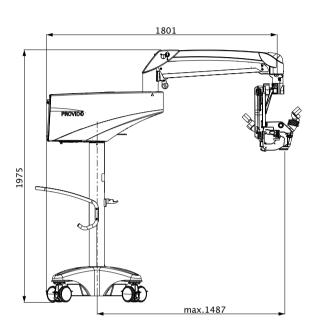

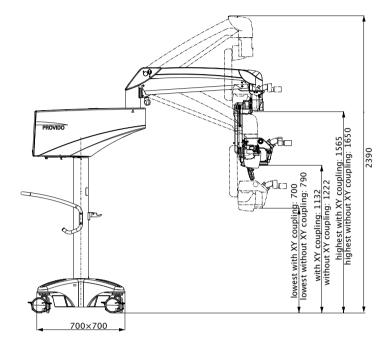

The application images used in this brochure were captured with a similar Leica surgical microscope. For all fluorescence modules, please check the status of regulatory classification & approval in your country with a local Leica Microsystems representative.



Leica Microsystems (Schweiz) AG Max Schmidheiny-Strasse 201 9435 Heerbrugg, Switzerland **(€ (€** ₀₁₂₃

Class I surgical microscope PROvido incl. accessories.

Class Ila Leica FL800 ULT.

Not all products or services are approved or offered in every market and approved labeling and instructions may vary between countries. Please contact your local Leica representative for details.

Leica Microsystems (Schweiz) AG \cdot Max Schmidheiny Strasse 201 \cdot CH-9435 Heerbrugg T +41 71 726 3333 \cdot F +41 71 726 3399

www.leica-microsystems.com

CONNECT WITH US!



